



# 1

## Inserting Batteries

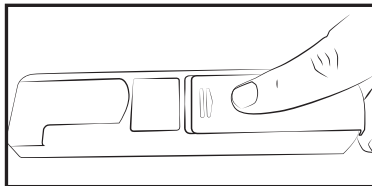

Open the battery cover by sliding it in the direction indicated by the arrows.

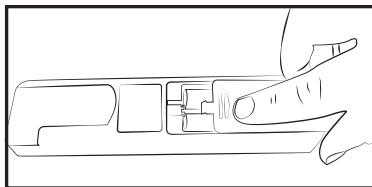

Insert two new AAA batteries as indicated by the polarity sign and then close the battery cover.

### CAUTION!

- DO NOT expose the unit to static electricity. Always disperse any static electricity from your body before holding the device.
- DO NOT insert batteries with their polarities reversed.
- DO NOT take measurements in a moving vehicle.
- DO NOT use a mobile phone during measurement.
- Contact your physician if you require assistance.
- The chest electrode will be impeded by excessive body hair, remove the hair at the electrode contact area when necessary.


**NOTE: Follow the User Guide for more detailed information.**

To access software for your device, please visit [www.theheartcheck.com](http://www.theheartcheck.com) for more details.


# 2

## Taking a Measurement

### Hand Measurement

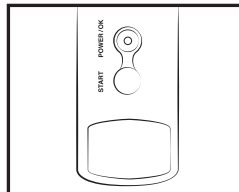


Press the POWER button to turn on the device.

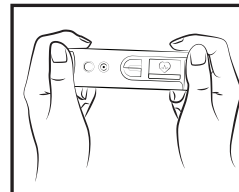


Hold the device as demonstrated in the figure above. Press the START button to take a measurement.

### Chest Measurement

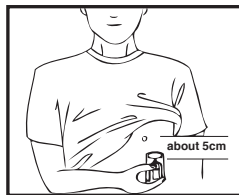


If a hand measurement is not ideal, use the device to take a chest measurement. Hold the device with your right hand and place the other electrode against the left side of your chest on bare skin about 5 cm (2 in) below the left nipple.

### Chest Measurement for Women

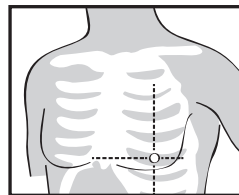

Position the device at the lower end of the breastbone and move horizontally to the middle of the left part of the chest. If necessary, lift the breast and place the left side electrode under the left breast.

# Quick Operation Guide

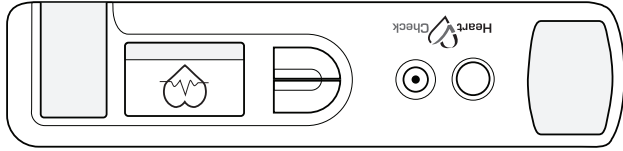

The HeartCheck™  
**Pen Device**

Powered by CardioComm Solutions, Inc.

Manufactured For:  
**CardioComm Solutions, Inc.**  
259 Yorkland Rd, Suite 200  
North York, ON M2J 0B5  
Tel: 1 (877) 977-9425 ext. 4 Fax: 1 (866) 576-4493  
[www.theheartcheck.com](http://www.theheartcheck.com)